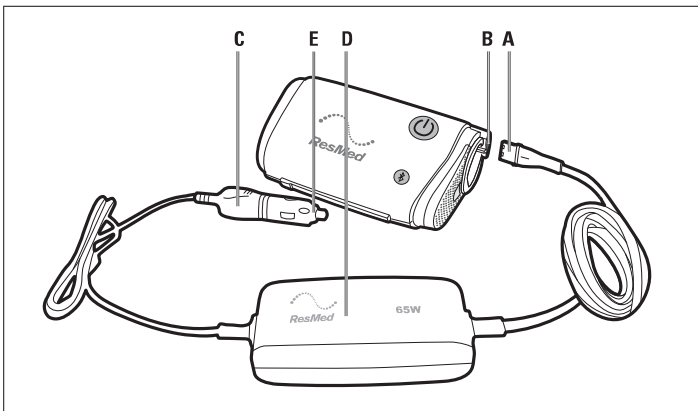
ResMed

AirMini™

DC/DC Converter 65W

User Guide

English | Deutsch | Français | Italiano | Español | Português | Nederlands
Svenska | Dansk | Norsk | Suomi | Česky | Eesti | Polski | Türkçe
简体中文 | 繁體中文 | 日本語 | العربية

ENGLISH

The AirMini DC/DC Converter 65W allows you to operate your machine from a vehicle cigarette lighter socket (12V or 24V DC power source) in a car, boat, or other vehicle equipped with a suitable battery.

The AirMini DC/DC Converter 65W is specifically for use with the AirMini machine.

Using the converter via a vehicle cigarette lighter socket

1. Position the converter on a level surface, with the ResMed logo facing up.
2. Plug the converter connector **(A)** into the machine power inlet/DC input socket **(B)**.
3. Plug the converter adaptor plug **(C)** into the vehicle cigarette lighter socket. The light **(D)** will glow green when it powers on.
If the battery is low, the light will glow amber. If the battery is too low to operate the machine, the converter and light will turn off.

Notes:

- Some vehicles require the ignition key to be turned to the “accessories” position to provide power to the vehicle cigarette lighter socket.
- The power cord is not replaceable.
- If the battery OR car, boat or other vehicle are not in good condition, there may not be enough reserve left in the battery to start the vehicle after use.
- When the converter is not in use disconnect it from the vehicle cigarette lighter socket to prevent battery discharge.
- The AirMini DC/DC Converter must only be used with the AirMini machine to ensure proper performance.
- ResMed recommends connecting a deep cycle battery to the converter to power your machine, as deep cycle batteries are optimized for frequent heavy discharges.
- If you need help or to report an unexpected issue, contact a ResMed representative, ResMed Technical Services or see www.resmed.com.

Cleaning

If necessary, wipe the outside of the converter with a clean, damp cloth.

Changing the fuse

1. Unscrew the fuse holder **(E)** from the top of the converter adaptor plug **(C)**.
2. Remove the old fuse and insert a replacement fuse of the same type and rating.
3. Re-screw the fuse holder.

WARNINGS

- Read instructions before use.
- Only use the converter when connecting to a 12V or 24V DC supply.

- Do not attempt to modify the converter. There are no user-servicable parts in the converter.
- Do not plug the power cord into a USB port.
- Make sure the power cord and plug are in good condition and the equipment is not damaged.
- Portable and mobile RF communications equipment should be used no closer to any part of the machine, including cables, than the recommended 10 cm separation distance.
- The machine should not be used adjacent to or stacked with other equipment. If adjacent or stacked use is necessary, the machine should be observed to verify normal operation in the configuration in which it will be used.

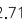
CAUTIONS

The Converter is intended for use on 12V or 24V negative ground systems (where the negative terminal of the battery is grounded). Using it on a 12V or 24V positive ground system may damage the converter and the vehicle's electrical system. If you are unsure whether you have a negative ground system, consult the car, boat or other vehicle manual.

- Replace fuse with same type and ratings to avoid risk of fire.
- Make sure the converter is not covered during use to avoid overheating the converter.
- The AirMini DC/DC Converter is not intended for use on an aircraft.

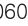
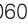
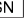
Note: For any serious incidents that occur in relation to this product, these should be reported to ResMed and the competent authority in your country.

Technical specifications

Length (connector to connector)	9.8 ft (3 m)
Fuse rating	12A/250V
Fuse type	Slow blow, high breaking capacity
Input	12V/24V  8/4A
Output	24V  2.71A
Operating temperature	+41°F to +95°F (+5°C to +35°C)
Operating humidity	10 to 95% RH non-condensing

Operating altitude	Sea level to 8,500' (2,591 m); air pressure range 1013 hPa (1013 cmH ₂ O) to 738 hPa (738 cmH ₂ O)
Storage and transport temperature	-13°F to +158°F (-25°C to +70°C)
Storage and transport humidity	5 to 95% RH non-condensing
Design life	5 years
Electromagnetic compatibility	Product complies with all applicable electromagnetic compatibility requirements (EMC) according to IEC60601-1-2, for residential, commercial and light industry environments. Portable RF Communications equipment (including peripherals such as antenna cables and external antenna) should be no closer than 10 cm to any part of the AirMini system. Otherwise degradation of the performance of this equipment could result. Information regarding the electromagnetic emissions and immunity of this ResMed device can be found on www.resmed.com , on the Products page under Service and Support.

Symbols which may appear on the machine

 Warning or caution, consult accompanying documents (IEC 60601-1);
 Class II equipment (IEC 60601-1); **IP21** Protection against insertion of fingers and against vertically dripping water (IEC 60601-1-11); **IP22** Protection against insertion of fingers and will not become unsafe when it is exposed to vertically dripping water with the device tilted up to 15 degrees (IEC 60601-1-11);  Direct Current (IEC 60601-1);  Fuse (IEC 60417-5016);  Manufacturer (ISO 15223-1);  **EC REP** European Authorized Representative (ISO 15223-1);  **RoHS** European RoHS;  **SN** Serial number (ISO 15223-1);  **10** China pollution control;  Humidity limitation (ISO 15223-1);  Temperature limitation (ISO 15223-1);  Operating altitude (ISO 15223-1);  **106** Atmospheric pressure limitation (ISO 15223-1);  Importer.  **MD** Medical device.
 See symbols glossary at ResMed.com/symbols.

Environmental Information (EU directive 2012/19/EE Waste Electrical and Electronic Equipment (WEEE))

The DC/DC converter should be disposed of separately, not as unsorted municipal waste. To dispose of your device, you should use appropriate

collection, reuse and recycling systems available in your region. The use of these collection, reuse and recycling systems is designed to reduce pressure on natural resources and prevent hazardous substances from damaging the environment. If you need information on these disposal systems, please contact your local waste administration. The crossed-bin symbol invites you to use these disposal systems. If you require information on collection and disposal of your ResMed device please contact your ResMed office, local distributor or go to www.resmed.com/environment.

Limited warranty

ResMed warrants that your ResMed DC/DC Converter 65W shall be free from defects in material and workmanship for a period of 1 year from the date of purchase by the initial consumer. This warranty is not transferable. If the product fails under conditions of normal use, ResMed will repair or replace, at its option, the defective product or any of its components.

This limited warranty does not cover: a) any damage caused as a result of improper use, abuse, modification or alteration of the product; b) repairs carried out by any service organisation that has not been expressly authorised by ResMed to perform such repairs; c) any damage or contamination due to cigarette, pipe, cigar or other smoke; d) any damage caused by water being spilled on or into the product.

Warranty is void on product sold, or resold, outside the region of original purchase.

Warranty claims on defective product must be made by the initial consumer at the point of purchase.

This warranty is in lieu of all other express or implied warranties, including any implied warranty of merchantability or fitness for a particular purpose. Some regions or states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply to you.

ResMed shall not be responsible for any incidental or consequential damages claimed to have occurred as a result of the sale, installation or use of any ResMed product. Some regions or states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from region to region. For further information on your warranty rights, contact your local ResMed dealer or ResMed office.

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